




BRUDER LAPAROSCOPY

SILASTIC RING APPLICATOR FOR FALLOPIAN TUBULAR STERILIZATION

Female sterilization by means of **Silastic Ring Applicator** is clinically well-established based on more than 20 years of experience. Our instruments are completely reusable and fully autoclavable. They are manufactured of the highest grade of materials by experienced and skilled craftsmen.

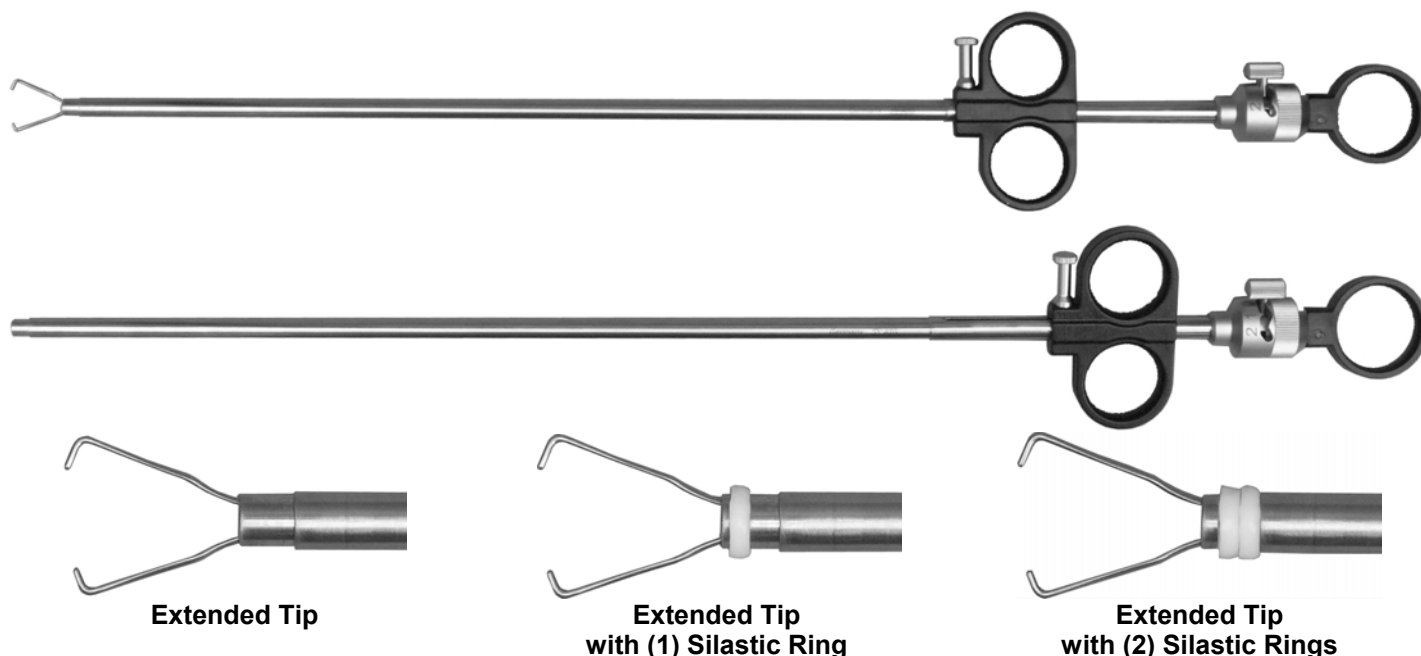

The **Silastic Ring Applicators** are made for single or dual ring application with one single insertion. The instruments are available in three different lengths for use through the operating scope (single puncture approach) or through trocar sleeve in conjunction with a laparoscope (multi-puncture approach).

List No. ARO105500	6 MM Ø, 40 CM SILASTIC RING APPLICATOR For single and dual ring application; use with operating laparoscope with 7mm Ø instrument channel (Suggest Bruder List No. ARO11006).
List No. ARO205501	6 MM Ø, 33 CM SILASTIC RING APPLICATOR For single and dual ring application; use with laparoscope and 7mm Ø trocar sleeve (See Bruder data sheet for appropriate sleeve).
List No. ARO208501	8 MM Ø, 27 CM SILASTIC RING APPLICATOR For single and dual ring application; use with laparoscope and 8mm Ø trocar sleeve (See Bruder data sheet for appropriate sleeve).
List No. ARO005500	SILASTIC RING (package of 10) FOR TUBAL LIGATION Non-sterile, autoclavable
List No. ARO005501	SILASTIC RING APPLIER AND GUIDE CONE SET (Note: (1) set comes with new instrument)

SILASTIC RING APPLICATOR FOR FALLOPIAN TUBULAR STERILIZATION

EFFECTIVE STERILIZATION PROCEDURE WITH SILASTIC RING APPLICATOR

A ten year study has proven the **Silastic Ring Applicator** to be one of the most reliable methods of mechanical tubal occlusion. Its high rate of success has contributed to this technique's worldwide acceptance.


BRUDER offers the greatest selection of instrumentation. This selection enables your physicians to choose from a variety of instrumentation including single or double puncture application.

This simple mechanical technique is the natural choice for safe, uncomplicated female sterilization. The Silastic ring is an inert, radio opaque silicone band that is simply applied to a looped section of the fallopian tube with the **Silastic Ring Applicator**.

The procedure provides high potential for reversibility. The success of surgical anastomosis is most dependent upon the skill of the surgeon.

ILLUSTRATION OF THE TUBAL LIGATION PROCEDURE:

1. Single puncture using 12 mm operating laparoscope.
2. Double puncture requiring a 10 mm laparoscope and 7 mm Ø trocar sleeve.
3. The **Silastic Ring Applicator** contains forceps to grasp the fallopian tube 3 to 4 cm from the cornual area.
4. The fallopian tube is gently drawn into the inner cylinder of the instrument forming a knuckle. The **Silastic Ring** located on the inner cylinder is released onto the knuckle, occluding the base. This procedure is repeated on the second fallopian tube. Once in place, the **Silastic Ring** will not slip.

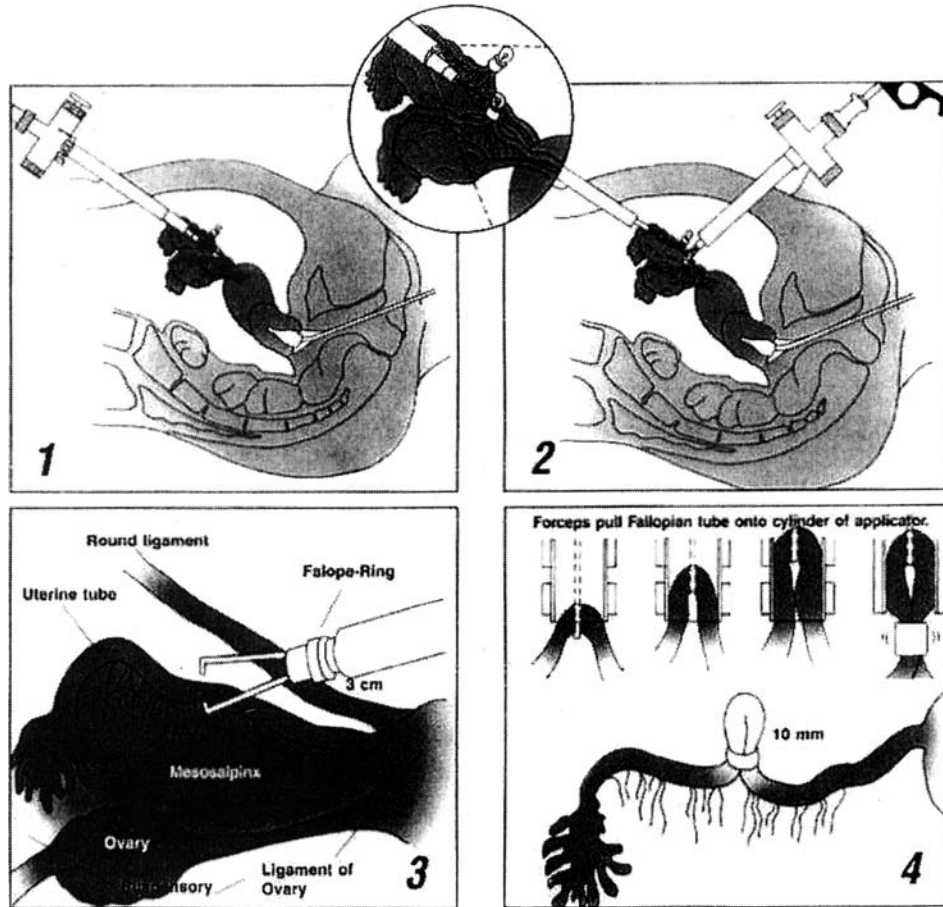

For a list of BRUDER laparoscopes and trocar sleeves used in conjunction with this instrument, request the appropriate data sheets.

Save and buy direct from the manufacturer.
We design, manufacture & sell the tools the surgeons use.

BRUDER specializes in the design and manufacture of new and innovative instruments, the repair of all competitive products and a fast turn-around time on repairs.



ARROW MEDICAL SUPPLY INC
Telephone Number: (847) 367-7771
Web Site: www.arrowmedical.com

28085 N Ashley Circle Bldg 105

Libertyville IL 600486
Fax Number: (847)367-7751
Email: info@arrowmedical.com